

Tissue Sampling Tags


Handling & Tagging Instructions

Please read carefully




1 Open bag with care

Check cutter is firmly attached to point of male tag

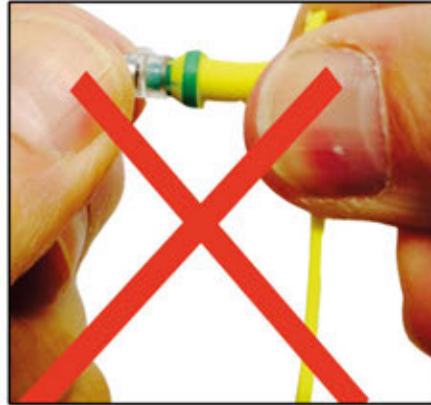


2 Do not remove cutter

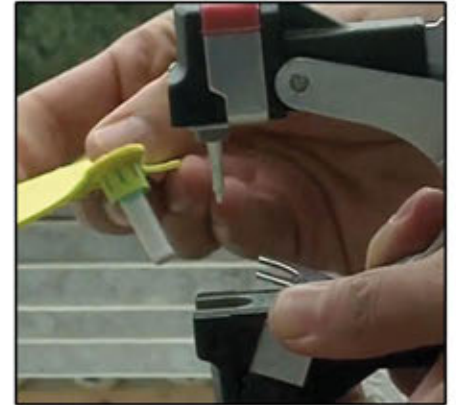

3 Load female first

Insert tube and secure tag below clip

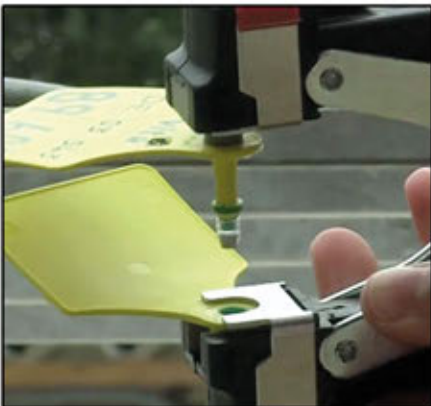

4 Load male second push onto pin

Ensure cutter is attached



5 Insert tag between two cartilage ribs

Close applicator fully
Audible click is heard!



6 Open applicator fully to release tag



7 Remove sample tube

Push tube gently up from below



8 Check tube is sealed and contains sample



9 Sealed tissue sample has same number as the tag